



EpiPen®/EpiPen Jr. Directions:

1. Identify student by first and last name
2. Check written order and label on medication
3. Remove EpiPen from yellow container/package
4. Remove gray cap from EpiPen
5. Hold the unit in your fist
6. Place black tip against upper outer thigh at right angle to leg
7. Press tip hard into thigh until activation
8. Hold in place about 10 seconds
9. Pull needle straight out of injection site
10. Massage injection site

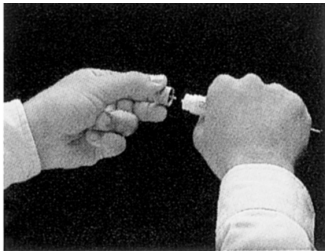

REMOVE CAP

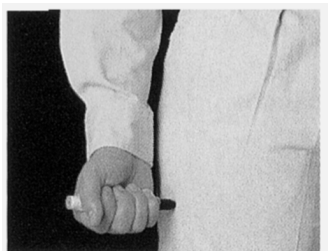

PLACE BLACK TIP
AGAINST THIGH



PRESS FIRMLY
HOLD 10 SECONDS

Additional tips about EpiPen:

There is one dose in each EpiPen

There are NO buttons to press, activation occurs by the pressure against the leg

Do not store at extremes of temperature, do not refrigerate

Do not confuse trainer (blue) with real device (yellow/white)

Check expiration date, but an expired one is better than nothing

Check window on device, fluid should be clear

EpiPen trainers can “click” but real ones do not

The device can go through clothing, but time usually permits clothing removal

Arrows show direction of injection

Take the EpiPen to the hospital for disposal, bend the needle back against a hard surface and carefully re-insert the fired unit, without replacing the safety cap—**NEEDLE FIRST**—into the amber carrying tube.

If insect sting, remove carefully by flicking, not squeezing (causes more venom to inject) and apply ice.



Provided by the Food Allergy Initiative, a national non-profit organization dedicated to finding a cure to life-threatening food allergies. For more information, please visit www.FoodAllergyInitiative.org or email Info@FoodAllergyInitiative.org